


addEASE® Binary Connector Activation with PAB® or 250mL EXCEL® IV Containers

Directions for Use: Activation

Use aseptic technique. Follow drug manufacturer's package insert and hospital protocol for preparation of medications. For use only with PAB and 250 mL EXCEL IV Containers. Pictorials for reference only. At time of use, check to ensure addEASE is firmly latched to container. If not firmly latched, discard assembly.

PAB Container

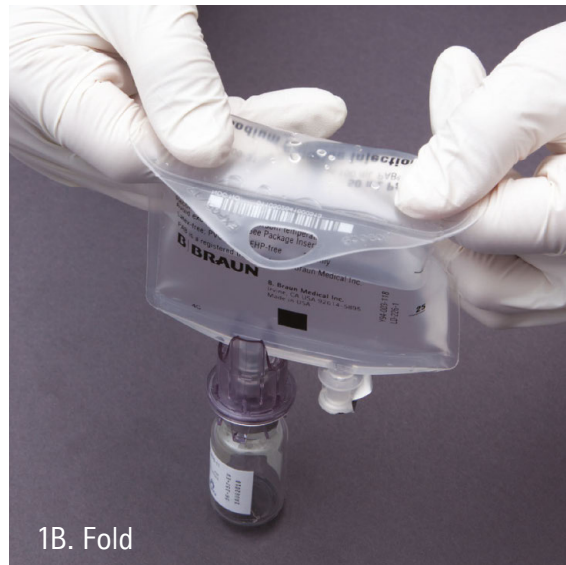

250mL EXCEL IV Container



1A./2A. To activate, position vial below container. For vials containing liquid drug, position vial above bag to activate.

1B./2B. Fold container at the fluid line.

1C./2C. Squeeze firmly to force plug from spike into vial. Repeat until vial is approximately half full. Do not overfill vial with liquid. Thoroughly mix to dissolve, following drug manufacturer's package insert.

CAUTION: Inspect solution and discard assembled system if cloudiness, haze or particulate matter is found. Consult medication vial directions for use for additional admixing instructions.

CAUTION: Do not disconnect medication vial from container. After use, dispose of assembled system per institutional protocol.

Refer to product labeling for complete instructions for use.